





1 YEAR WARRANTY
warranty@adjust.fr

MONOBLOC KIT ASSEMBLY KIT FOR MONOBLOC SEAT


The kit is comprised of a black Velcro strip 5cm wide:
50cm of hook section attached to the 1.5m loop section.

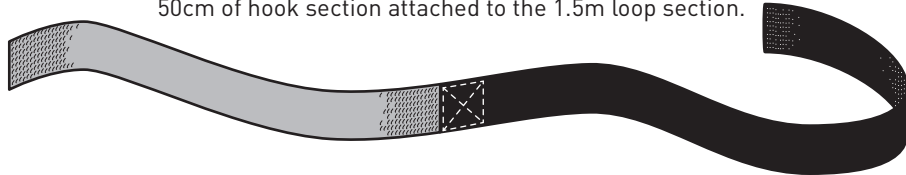


1 Attach the Velcro hook part to very top back of the back part of the lumbar cushion, leaving an equal length hook part on each side.

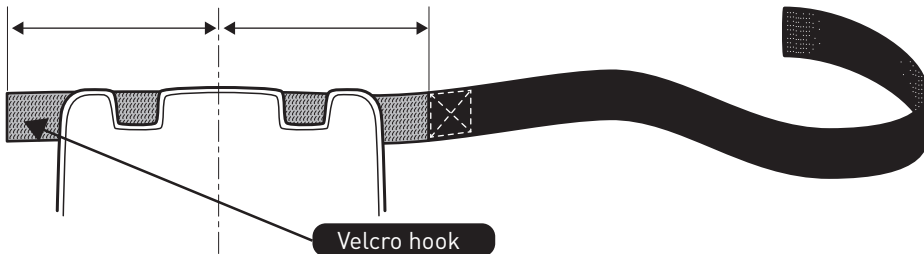

2 Place the lumbar cushion against the back of your chair at the right height (ensure you leave at least 10cm at the bottom of the back piece that you will subsequently slide between the seat and the back of the chair)

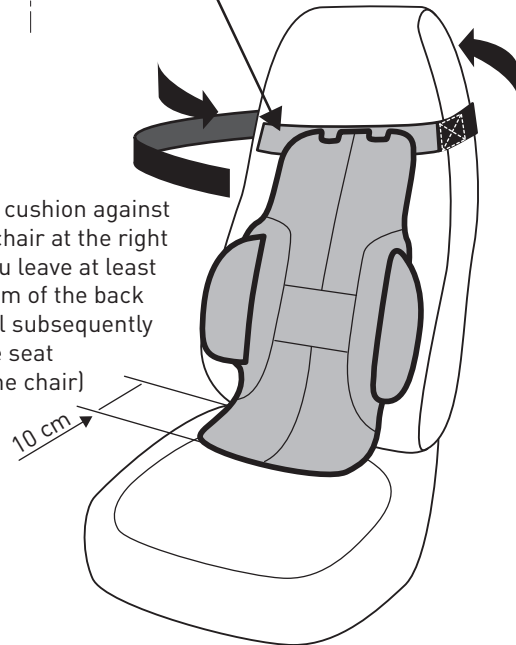

3 Once positioned at the correct height, work the loop strip around the back of the chair, stretching it, and attach it to the first section of the hook Velcro.

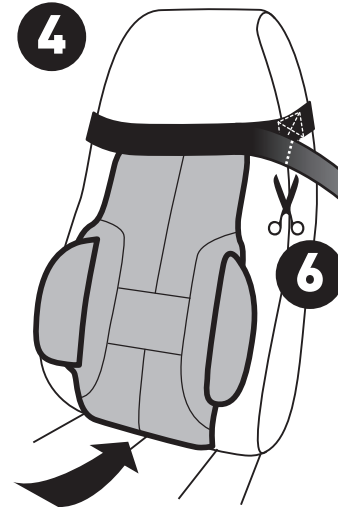

Then slide the loop strip along the front of the back piece and attach it to the second hook area.

This serves both to hide the cut outs needed for fastening in the case of headrests with rods, and to ensure it is securely fastened.

5 Finally, slide the bottom of the back piece between the base and the back of the chair.

6 When you are sure it is at the correct height and adjusted correctly, cut off the excess part of the hook strip.


ALL PRODUCTS **ad'just**



LUMBAR SUPPORT CUSHION
Lumbar and lateral support

MONOBLOC KIT
Kit for assembling the lumbar support cushion on a monobloc seat

SEAT CUSHION
Adjust the angle of the back / leg base to alleviate tension

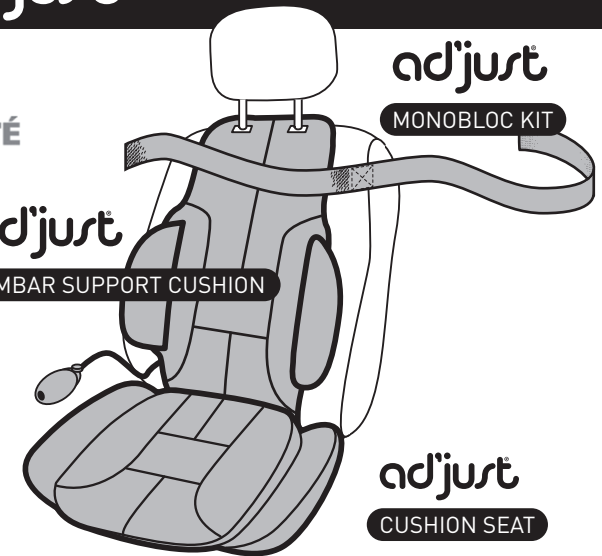

contact@adjust.fr

www.adjust.fr



www.adjust.fr/youtube

adjust sas : 92 route de Jallieu | 38080 Saint Marcel Bel Accueil | France

® Patent and registered trademark