



ASSEMBLY KIT FOR MONOBLOC SEAT

The kit is comprised of a black Velcro strip 5cm wide:
50cm of hook section attached to the 1.5m loop section.

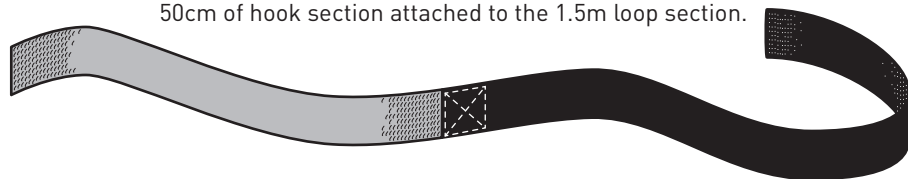

- 1 Attach the Velcro hook part to very top back of the back part of the lumbar cushion, leaving an equal length hook part on each side.

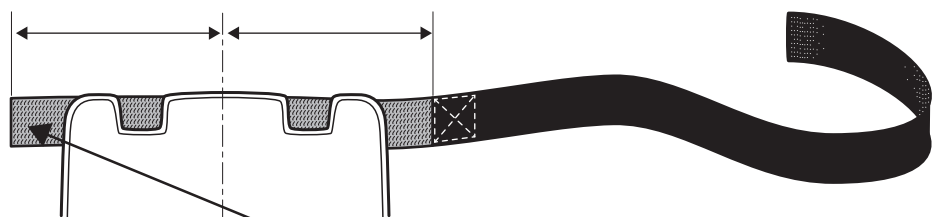


Velcro hook

2

Place the lumbar cushion against the back of your chair at the right height (ensure you leave at least 10cm at the bottom of the back piece that you will subsequently slide between the seat and the back of the chair)

10 cm

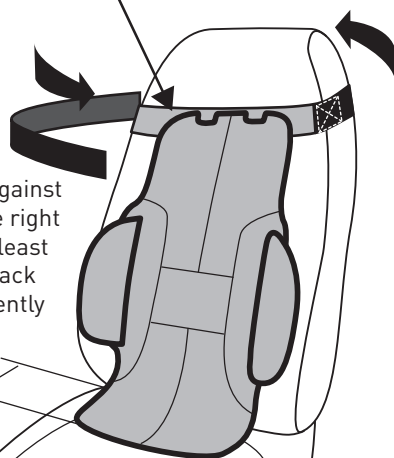

3

Once positioned at the correct height, work the loop strip around the back of the chair, stretching it, and attach it to the first section of the hook Velcro.



2 YEARS WARRANTY


warranty@adjust.fr

EN

4

Then slide the loop strip along the front of the back piece and attach it to the second hook area.

This serves both to hide the cut outs needed for fastening in the case of headrests with rods, and to ensure it is securely fastened.

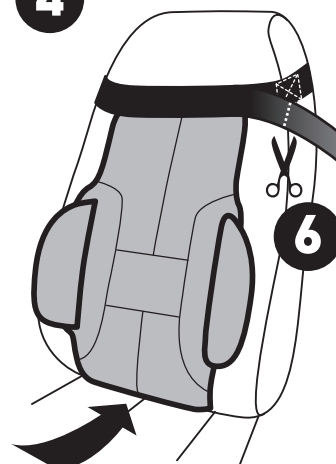

6

When you are sure it is at the correct height and adjusted correctly, cut off the excess part of the hook strip.

5

Finally, slide the bottom of the back piece between the base and the back of the chair.

ALL PRODUCTS **adjust**



Préventica SANTÉ / SÉCURITÉ
CONSEILS // SALONS AU TRAVAIL
PRIX DE L'INNOVATION
LILLE EUROPE 2016

LUMBAR SUPPORT CUSHION
Lumbar and lateral support

MONOBLOC KIT
Kit for assembling the lumbar support cushion on a monobloc seat


SEAT CUSHION
Adjust the angle of the back / leg base to alleviate tension

adjust

LUMBAR SUPPORT CUSHION

adjust

MONOBLOC KIT

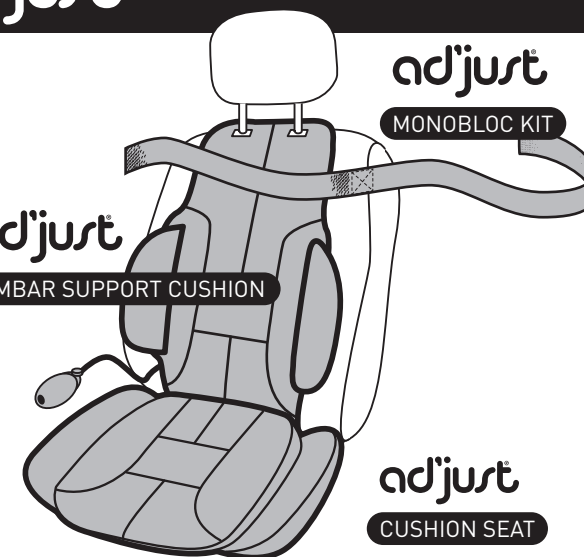

adjust

CUSHION SEAT



contact@adjust.fr

www.adjust.fr

YouTube

www.adjust.fr/youtube



adjust sas : 92 route de Jallieu | 38080 Saint Marcel Bel Accueil | France

® Patent and registered trademark